

## ONSET® CARTRIDGE CONNECTOR

### DEVICE DESCRIPTION:

The Onset Cartridge Connector (hereinafter “Connector”) is used for the transfer of sterile fluids from one sealed container into a second sealed container (the containers are not provided). The Connector provides a reservoir for collecting waste fluid displaced from the second container during the transfer process.

Each Connector is individually packaged and sterilized using gamma irradiation. The Connector includes a sterile Transfer Tube that creates a fluid path between two containers. The Connector also contains a sterile Exhaust Tube that connects the second container with the Connector’s Fluid Reservoir. The Connector is designed to allow the solution from the first container to be delivered via the Transfer Tube into the second container, where it mixes with the fluid in the second container. Simultaneously, fluid is displaced from the second container through the Exhaust Tube into the Fluid Reservoir, which holds up to 1.7 mL of fluid.

### INTENDED USE AND INDICATIONS FOR USE:

The Connector is a sterile device used for transfer of sterile fluids and collection of displaced fluids into a reservoir.

### CONTRAINDICATIONS:

The Connector is not indicated for direct line administration of solutions to any patient or user.

### DIRECTIONS FOR USE:

Use aseptic technique when opening the sterile packaging and during handling of the device in order to avoid contamination of the sterile fluid path. Remove the Connector’s Safety Cap prior to use. This device may be used alone or with the Onset Mixing Pen when mixing two solutions together.

### PRECAUTIONS:


Use only as directed.

Inspect packaging prior to use. Do not use if sterile packaging barrier has been compromised. Do not use the connector if damaged or appears to be damaged.

Take care to avoid contact with the Transfer Tubes, as they are sharp and may result in needle stick injury if contacted. Dispose of the Connector in a sharps container after use.

### HOW SUPPLIED:

The Connector is supplied sterile, individually packaged, for single use.

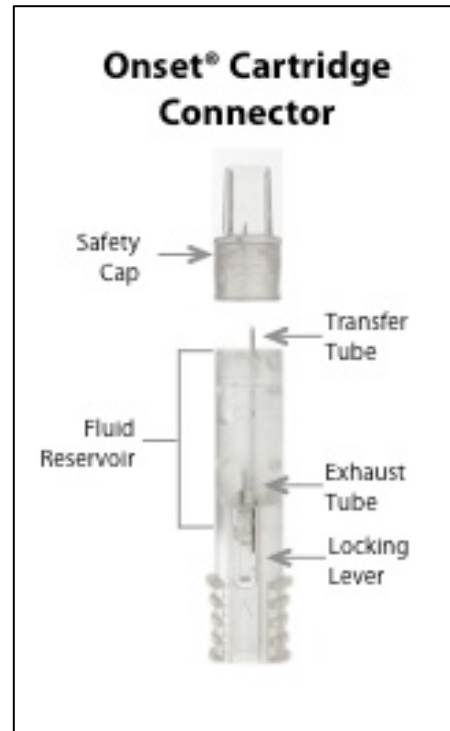

### STORAGE AND HANDLING:

There are no special storage requirements for the Connector. Store at room temperature between 18 and 25 degrees C (64 and 77 degrees F).

### CUSTOMER CARE AND QUESTIONS:

Call 1-877-336-6738 for more information, questions or support regarding the Connector or the referenced Onset Mixing Pen.

Federal law (U.S.A.) restricts this device to sale by or on the order of a medical practitioner licensed by the law of the State in which he/she practices to use or order the use of the device.

Symbol					
Definition	Caution, consult accompanying documents	Do not use if package is damaged	Sterilized using irradiation	Do not re-use	Consult instructions for use

Manufactured for Onpharma Inc., Los Gatos, CA 95030  
 By Phoenix DeVentures Inc., Morgan Hill, CA 95037  
 Customer Care (877) 336-6738